



1 LOAD Endoscope

- Press OPEN/CLOSE LID button or press foot switch to open/close lid.
- Place endoscope in basin with control head in the lower left corner.
- Locate the appropriate Hook-up Connector Block.

2 ATTACH Endoscope

- Engage Hook-up Connector Block with Basin Connection Manifold.
- Connect tubing to endoscope by matching description on Hook-up Connector Block to correct location on endoscope.
- Verify tubing is not kinked or pinched.
- Connect the channel separator if applicable.

3 ENTER ID Data and Reprocessing Program

- Press the MENU button.
- Using the Barcode Scanner, scan the operator's barcode.
- Select "Endoscope" from the Main Menu and press the O.K. button. Scan the barcode on the Hook-up Connector Block.
- Scan the barcode on the endoscope.
- Enter the patient number and scan the physician barcode (as applicable). Confirm each step by pressing O.K.

4 START Reprocessing Cycle

- The lid will close and cycle will start automatically following the last scanned item.

5 TEST Disinfectant Concentration

- At the end of the cycle, when prompted, test the Minimum Recommended Concentration (MRC) with a RapiCide[®] PA test strip of the disinfectant sample taken from the Sample Port.
- Press the START button if MRC passed or STOP if MRC failed to finish cycle.

6 REMOVE Endoscope

- Cycle is complete when reprocessor indicates on monitor that lid may be opened.
- Press the OPEN button and scan the operator's barcode. Lid will open.



Control Panel			
	Open/Close Lid		Start
	Menu		Stop
	O.K.		Scroll Up
	Cancel		Scroll Down

Weekly Service

1. Check connector blocks for wear
2. Lubricate and inspect manifold O-rings
3. Clean and inspect basin screen
4. Run water line disinfection

Refer to User Manual for more detailed instructions
1-800-444-4729
www.minntech.com/medivators